

MK-000A

D+Lady

***User's
Manual***

Thank you for purchasing our product MK-000A Doctor Lady (Dr. Lady)
This product is a medical device. Please read this manual carefully before use.
This manual includes warranty certificate for the quality. Please keep this carefully to prevent damage or being lost.

Please use this product only for therapeutic purpose.

[Information to be written in accordance with the Medical Device Act]

Device name: Combination stimulator for Level 3 medical purpose / electrical neuromuscular stimulator for non-implantable urinary incontinence

Product name: Dr. Lady

Model name: MK-000A

Product license number: No. 14-1099

Manufacturer: Buheung Medical

Manufacturing license No.: No. 699

Manufacturing Country: Republic of Korea

Manufacturer address: 176 Beukot road, Geumcheon-gu, Seoul, Korea

Manufacturer contact number: Tel) 02-838-5866 / Fax) 02-838-5334

Date of manufacturing and production number: Indicated separately

Salesperson information:

Salesperson	
Registration number for medical equipment distribution	
Address of the salesperson	
Contact number	



[Product configuration]


Please check product configuration before use of the product at the first.

- ◆ Product Main Body
- ◆ Vagina insertion type probe
- ◆ Adapter for charging
- ◆ Manual
- ◆ Pouch for exclusive use


1. [Name for each part of the product]

Name for each key part of the product are as follows.

1) Main body of the product

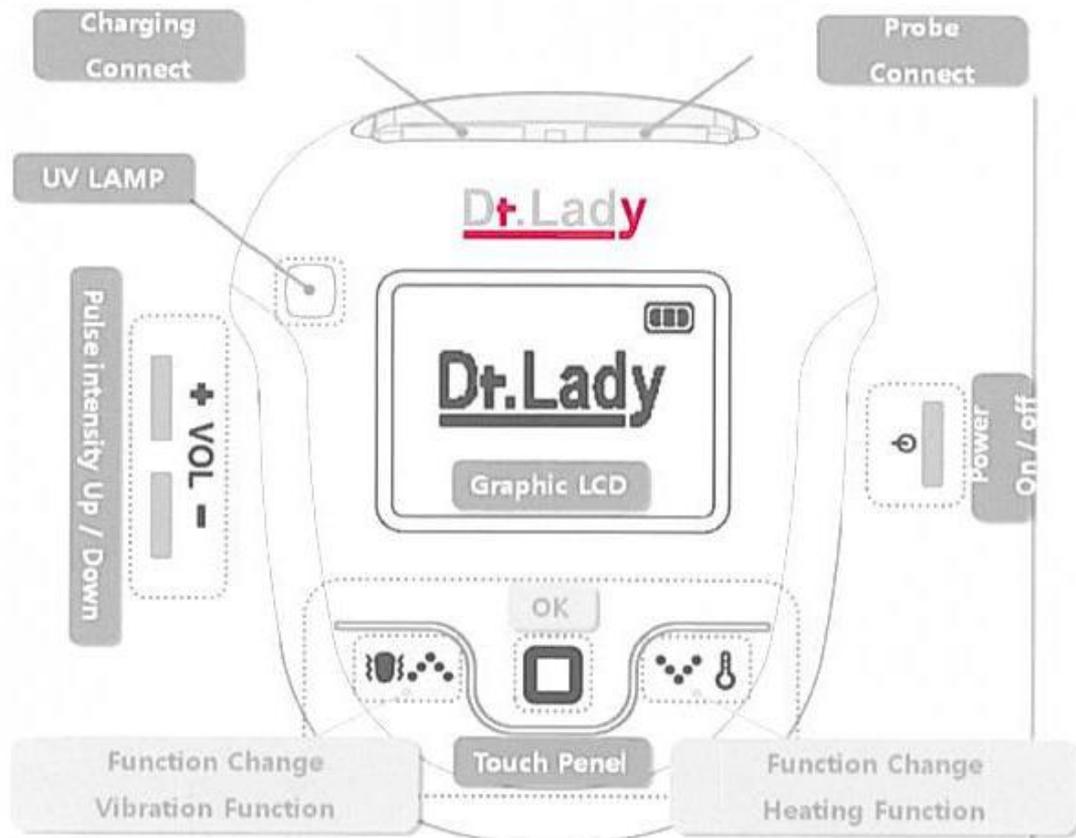

< Front side of the main body of the product >

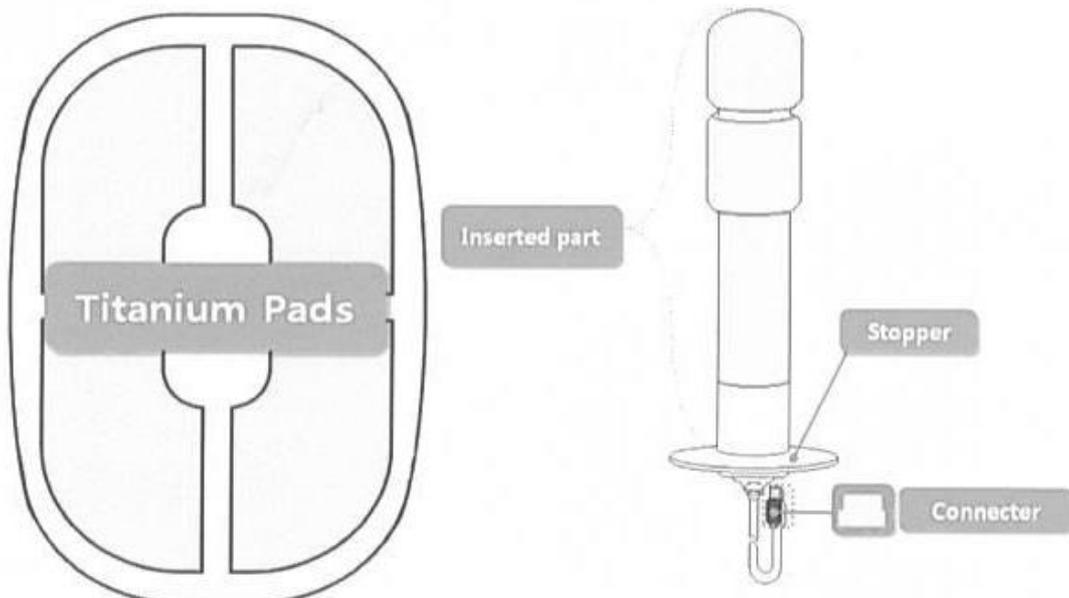

< Back side of the main body of the Product >

< Vagina insertion type probe >

2. [Product charging]

In case of using this product for the first time, use this product one to two hour after charging the product following the charging method mentioned below.

- ① Connect the adapter for charging with operating device.
- ② When adapter is connected to charging socket, 「charging...」 sign is displayed and charging starts.
- ③ Balance of battery is displayed on upper right side of the screen whenever power supply of the product is turned on.

[Notice]


- ◆ Be careful not to be confused with charging connector area and probe connector area.
- ◆ Charging connector area is located in the upper left corner of the main body of the product.
- ◆ Charging adapter specification: Micro USB 5 pin, 5V 1000mA



3. [Power On / Off of the product]

This is the process to supply power to use the product.

- ① Press power button on the right side of the operating device for about 3 second.
- ② When power is turned, home screen is shown as Right.
- ③ When □ is touched, it moves to next step.
- ④ Whenever power button is pressed for long during the operation of the product, power is turned off.

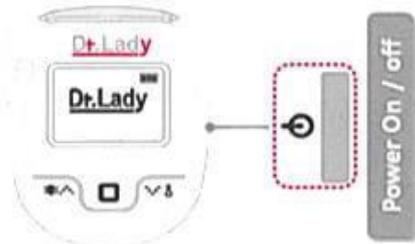

[Note]

Whenever power button is pressed for short during the operation of the product, operation stops and moves to Home screen.

When there is no operation for certain period of time while power is turned on, power of the product is turned off automatically.

(Home Screen - 5 seconds/ other screen - 30 seconds)




4. [Language Setting]

If you are using it for the first time after purchase, you will see "Language Settings" screen as follow:

Select the language you want by utilizing  and  on touch pad area and touch □.

- ※ The screen on the right side is an example when English language is selected.

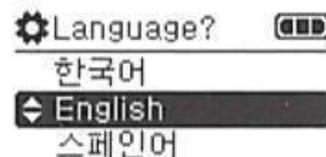

[Note] Touch pad area works smoothly even when it is touched slightly with finger.


5. [New User Registration]

You can register up to 3 users.

New user registration can be done by adding new user. If you want to select previously registered user, please check the way how you can make 「Existing user selection」.

① When is pressed in Home Screen, You can see the user screen as BELOW.

(When using the first time, it should be done after language setting.)

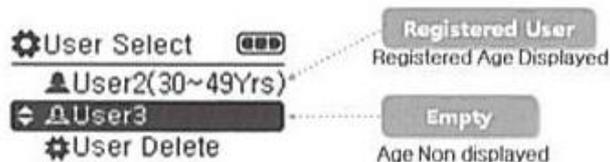

② Select the user by utilizing and and touch .

(In the example screen shown above, user registration can be done at 「user 3」.

③ It will moves to 「age setting」.

[Note]

In case you want to select existing registered user, please refer to the method 6. 「Existing user selection」.


In case you want to delete existing user and set new user, please refer to the way 12. 「User delete」.



6. [Existing user selection]

【Existing user selection】 is the process to select the user who already finished registration.

Select the user by utilizing and at the **right** screen and touch .

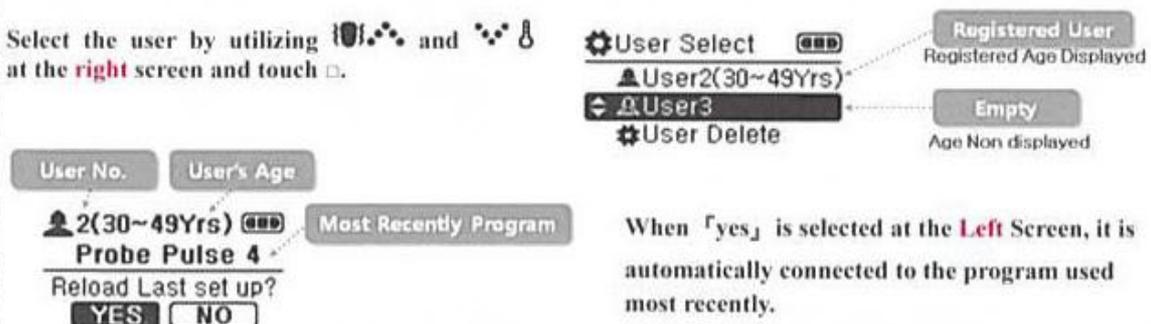

[Note]

In case you want to add a new user, please refer to 4. 「user registration」.

In case you want to delete existing user and set new user, please refer to 12. 「User delete」.



7. 【Age setting】


This setting needs to be done only one time when new user registration is done.

When age setting is completed, whenever it is used, the age which was already set appears at the right side of the 「User number」 on the user selection screen.

[Note]

◆ User age setting is a very important setting to use each function. Please make sure to use after setting the age of the user, and do not use with different age setting for your age.

◆ In case wrong age setting is done, delete the relevant user and start the process again from 4. 「New user registration」.

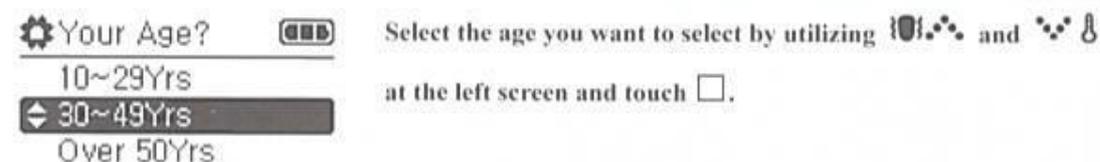

8. 【Program selection】

This product is composed of six probe program and two pad program.

Probe Pulse 1	Probe Pulse 2	Probe Pulse 3	Probe Pulse 4	Probe Heater	Probe Massage
Incontinence	Sexual Disfunction	Vagina Muscle Exercise, Kegel Exercise	Genital Cold, Atony, Myoma, other woman Diseases	Blood Circulation, Cancer, Heating	Nerve Exercise, Fine Vibration


Pad Pulse 1	Pad Pulse 2
Menses Pain Mid-Freq.	Menses Pain Electro Acunpture

【Things to be noted when selecting program】

◆ In case 「Probe program」 is selected without being connected to main body, 「Probe Disconnected」 sign appears. Please Connect the Probe to operating mechanism and touch [confirm icon].

◆ In case 「Pad program」 is selected while main body is connected to probe, 「Disconnect the Probe」 sign appears. Separate the probe from operating mechanism and touch [confirm icon].

- ① Select the program you want to select by utilizing [up/down arrow icons] and [confirm icon] in the screen above and touch [confirm icon].
- ② When the text appears, execute the request relevant to the text and touch [confirm icon].
- ③ When program selection is completed, relevant program is executed.

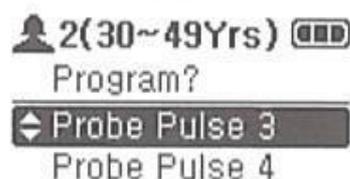

9. [Program execution]

This is the process to execute the program set before and there are a few preparation processes before use. These processes can take few seconds or minutes depending on the selected program or the external environment.


◆ When program starts, intensity of the frequency is "0" level. Adjust intensity of the Pulse with the intensity adjusting button on the left side of the operating mechanism and use.

[Reference] 10. [How to adjust frequency intensity]

◆ When is touched during execution, the Red and White flashing and it becomes pause mode. When is pressed again in pause mode, the program under execution starts to be executed again.


1) Probe program

- ① When words appears on the first screen in the right side, check the probe connection and touch .
- ② When probe is connected, the phrase "Please Wait." is displayed.
- ③ When warming up is completed, the phrase "Put the probe in your body." is displayed.
- ④ When the phrase " Put the probe in your body." appears, Insert the probe into vagina and touch the to execute program.

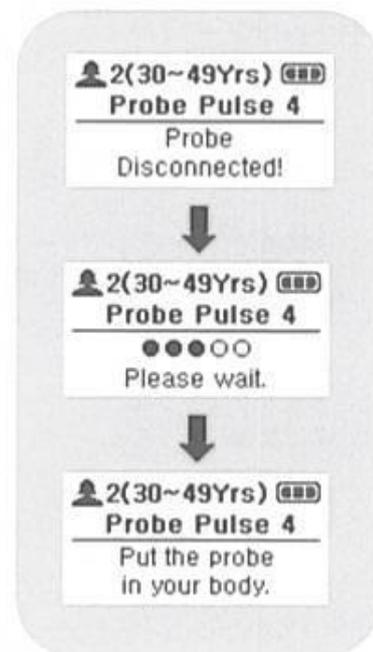

[Note]

- ◆ Check the damage, broken or unusual discolor, etc. of probe before use or insert into Vagina.
- ◆ Avoid to use the probe if there is the damage, broken or unusual discolor, etc.
- ◆ Probe can only be inserted into vagina of woman. Never insert into anus or other part of the body.
- ◆ probe is personal. Never use a single probe for a number of people.
- ◆ Insert probe to the end. Unexpected pain, fall out, inconveniency, etc. can happen and the effect of use can be reduced. Insert the bottom plate of the probe to the end so that it is completely attached to vulva.
- ◆ When probe is inserted put enough water or water soluble gel on the insertion part of the probe.
- ◆ Do not use foreign substance such as condom or femidom on the probe.
- ◆ When removing the probe from the body during the use of probe program, make sure to turn off the power of this product before taking out.
- ◆ Avoid movement as much as possible during the use of probe program.

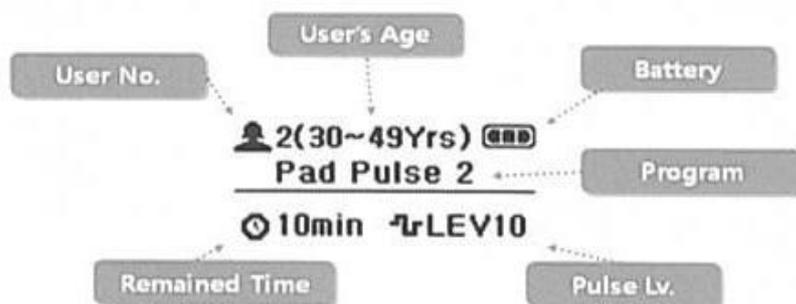

[Note]

- ◆ In case Pad program is executed, put enough amount of gel on the skin where you want to use pad program. If not, pain can be generated during the use or effect can be reduced or in some cases, damage can be caused. In addition, product may not recognize the attachment state.
- ◆ When you want to remove pad from your body during the use of pad program, make sure to turn off the power of the operating mechanism before removing.
- ◆ During the use of pad program, avoid movement as much as possible.



10. 【Frequency intensity adjustment】

Frequency intensity adjustment function is the function to adjust intensity of the frequency generated from each program.

Proper intensity can vary for each time of use depending on each individual and use environment or number of use.

① As shown below, frequency intensity control button is located on the left side of the product main body.

② All programs are set at the frequency intensity 「Lv. 0」 (zero level) at the start of the program.


③ Frequency intensity can be set up to 20 levels of intensity.

④ Frequency intensity increases one level each time when 「+」 button is pressed for short time and five levels each time when it is pressed for long time.

⑤ Same as 「+」 button, one level for each time when 「-」 button is pressed for short time, and five levels for each time when it is pressed for long time.

⑥ Frequency intensity is displayed in red square on the right side of the screen.

⑦ Adjust the frequency intensity properly by pressing frequency intensity adjusting button and use.

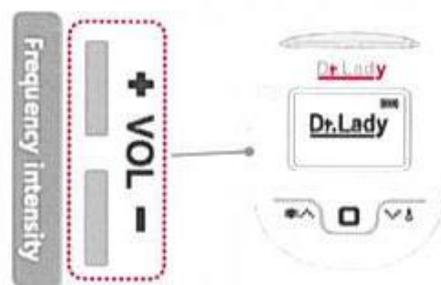

[Note]

- ◆ Adjust the intensity to the level which is not a burden for use.
- ◆ Intensity which is higher than necessary intensity can cause muscle pain or discomfort.

7. 【Age setting】


This setting needs to be done only one time when new user registration is done.

When age setting is completed, whenever it is used, the age which was already set appears at the right side of the 「User number」 on the user selection screen.

[Note]

◆ User age setting is a very important setting to use each function. Please make sure to use after setting the age of the user, and do not use with different age setting for your age.

◆ In case wrong age setting is done, delete the relevant user and start the process again from 4. 「New user registration」.

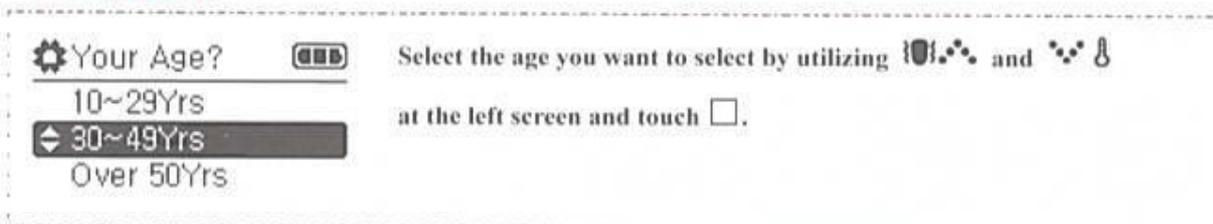

8. 【Program selection】

This product is composed of six probe program and two pad program.

Probe Pulse 1	Probe Pulse 2	Probe Pulse 3	Probe Pulse 4	Probe Heater	Probe Massage
Incontinence	Sexual Disfunction	Vagina Muscle Exercise, Kegel Exercise	Genital Cold, Atony, Myoma, other woman Diseases	Blood Circulation, Cancer, Heating	Nerve Exercise, Fine Vibration


Pad Pulse 1	Pad Pulse 2
Menses Pain Mid-Freq.	Menses Pain Electro Acunpture

【Things to be noted when selecting program】

◆ In case 「Probe program」 is selected without being connected to main body, 「Probe Disconnected」 sign appears. Please Connect the Probe to operating mechanism and touch [confirm icon].

◆ In case 「Pad program」 is selected while main body is connected to probe, 「Disconnect the Probe」 sign appears. Separate the probe from operating mechanism and touch [confirm icon].

- ① Select the program you want to select by utilizing [up/down arrow icons] and [confirm icon] in the screen above and touch [confirm icon].
- ② When the text appears, execute the request relevant to the text and touch [confirm icon].
- ③ When program selection is completed, relevant program is executed.

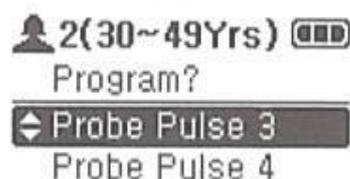

⑤ When the program starts, probe vibrates number of times and executes the program.

⑥ When  or  is touched during the execution of probe program, vibration or heating function is performed together.

Vibration function and intensity adjustment Heating function and temperature control



◆ In case of 1-4 of the probe program, heating function can be set up to maximum 2 step.

⑦ The screen executing probe program is as follow.



2) Pad program

① In case of the pad program, if the probe is connected, the phrase 「Disconnect the Probe」 appears.

Remove the probe from main body of the product and press .

② If probe is not connected, the phrase 「Put the electrodes of controller on your skin」

③ When is pressed after putting pad on the skin, program starts to be executed.

④ The screen during pad program execution is as follow.


 2 (30~49Yrs) 

Pad Pulse 2

Put the electrodes of controller on your skin.

11. 【Program ending】

- ① When the time set for each program is over, buzzer sounds and program stops automatically and screen moves to home screen.
- ② When power button is pressed for short time during program execution, program ends and screen moves to home screen.
- ③ When power button is pressed for long time during program execution, program ends and power is turned off.

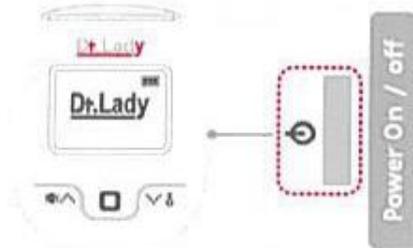


[Note]

- ◆ During the execution of program, Press power button and end program before removing the probe from vagina or taking off the pad from skin.
- ◆ Intensity which is higher than necessary intensity can cause muscle pain or discomfort.



11. 【UV lamp function】

- ① On the left front of the operating mechanism screen, there is UV lamp.
- ② Put probe and operating mechanism in designated product pouch.
- ③ Press 「-」 for about 3 seconds of the frequency intensity adjustment buttons at power off state.
- ④ When UV lamp is lit, close the designated pouch and zip it.
- ⑤ UV lamp is turned off automatically after 15 minutes.

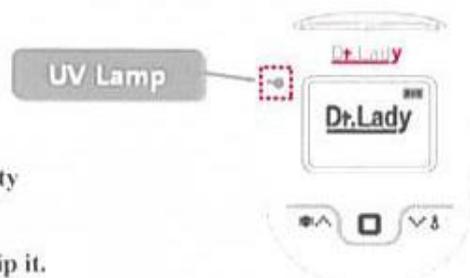


[Note]

- ◆ If possible, do not open the product pouch while operating UV lamp function.
- ◆ Make sure to use designated product pouch when using UV lamp function.
- ◆ Do not look at the UV lamp directly.
- ◆ Use UV lamp only within the product pouch

12. 【User delete】

Delete the user registered previously. New user can be added in the area used for deleted user.

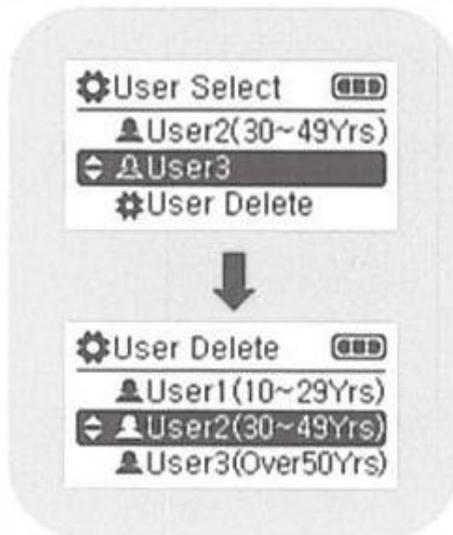

① Select 「User Delete」 at the 「User Select」 screen in the left first area by utilizing  and  and touch .


② Select the user to be deleted in the 「User Delete」 screen in the left second area and touch to delete the user.

③ Age of the deleted user does not appear at the right area of the user as shown in 「user 3」 in the left last screen.

④ New user can be added in the area used for deleted user, and adding can be done according to 【New User Registration】 method mentioned previously.



14. 【Water Proof Part of Probe】

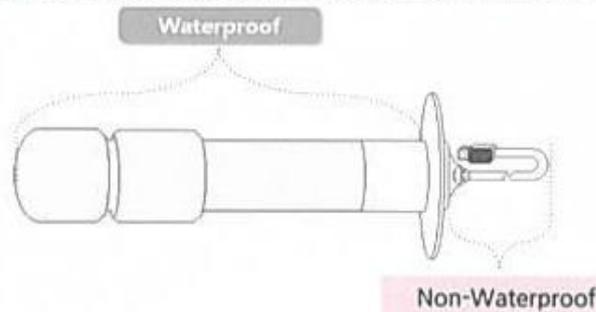

[Before A/S request]

- ◆ Power cannot be turn on.
 - Was the product fully charged?
 - Was the power supply of the product pressed for about 3 seconds?
 - Charging adapter needs to be checked. (Charging adapter specification: Micro USB 5 pin, 5V 1000mA)
(Please use charging adapter with caution to avoid damage to the charging adapter.)
- ◆ Probe program does not work.
 - Was the product fully charged?
 - Was the probe fully fastened to the product main body?
 - Was the frequency intensity adjusted after probe program started?
- ◆ Pad program does not work.
 - Was the product fully charged?
 - Is the probe not connected to the product main body?
 - Was the frequency intensity adjusted after probe program started?
- ◆ Button (or touch pad) does not work.
 - Might be a transient error. Turn off the power and wait more than 1 minute and turn on the power again.



[Items need to pay attention on Use]

- ◆ This product is a medical device. Do not use in case of the case mentioned below.
 - ① In case you are a pregnant woman or a woman recently delivered baby, or in case you have a history of serious medical treatment for feminine disease
 - ② Children under 10 years of age or elder over 80 years of age
 - ③ In case you have artificial organs such as artificial heart (Peacemaker) or artificial joint
 - ④ In case you have disease such as psychosis, epilepsy, etc
 - ⑤ In case you have the history of heart disease
 - ⑥ In case you have disease similar to those mentioned above or history of serious physical or mental illness
- ◆ Avoid using the product at the area near to fire source or heating device.
- ◆ Do not put the product into the water or use the product near to moist area.
- ◆ Do not use this product near to home appliances with high-current or high-voltage.
- ◆ Do not apply a strong shock to the product.
- ◆ In case abnormal state occurs such as noise, abnormal heat during the use of this product, stop using this product immediately and contact manufacturer or the place you bought this product.
- ◆ Do not use on damaged skin.

[Items need to pay attention for storage]

- ◆ Clean the Pad or probe of this device by gauze with alcohol after use and dry it immediately and keep it.
- ◆ In case the main body of the device is dirty, wipe it with gauze.
- ◆ Do not clean this device by using chemical such as thinner, benzene or acetone.
- ◆ Do not keep this product near to home appliance with high current and high voltage.
- ◆ Do not keep this product near to fire source or heat generating device.
- ◆ Do not put this product into water or keep it in moist place.
- ◆ Do not apply a strong shock to the product.
- ◆ Repair should only be done at the manufacturer or the place which acquired designated training or the certificate from manufacturer.
- ◆ Do not disassemble the product.
- ◆ Make sure to keep the product in designated pouch after use.
- ◆ This product must be cleaned before and after use of this product. Use of contaminated product may cause side effects such as infection, etc.

[Inquiry for A/S]

This product is a product supplied to customers after thorough quality inspection process and confirmation but abnormality can happen by chance. Please contact manufacturer or dealer mentioned below before asking A/S.

Manufacturer		Saler
Buheung Medical	Company name	
02-838-5866	Phone number	
176 Beukot road, Geumcheon-gu, Seoul, Korea	Address	

When contacting A / S, depending on the cause of product failure, repair costs can be generated even within the product warranty period and, in case of consumable items, free repair and replacement may not be available.

If you request A / S through manufacturer or dealer, please follow the A/S request procedure mentioned below.

- ◆ In the Quality warranty sheet of the product, all the information needs to be contained such as customer information, place of purchase, date of purchase and product number, etc. should be written.
- ◆ In case you leave the product in the manufacturer and dealer for repair, leave it in clean state. Product can get damage or destruction due to foreign substance, etc.
- ◆ In case you send the product via courier, make sure to put name, address and contact point correctly.

Product Warranty Sheet

We guaranty the quality of our product for 1 year after your purchase of the product according to our customer regulations. Repair cost after the warranty period will be treated by actual cost incurred to us in accordance with our headquarter regulation. Product failure due to customer negligence will not be compensated.

Product information	Product Name: MK-000A Doctor Lady (Dr. Lady)	
	Product number (S/N)	
Customer information	Customer name:	Phone number:
	Address:	
Dealer information	Dealer name:	Phone number:
	Date of purchase:	Sales man name: (signature)
Others		

[Free Service]

- ◆ You can get free repair for the failure due to product failure within 1 year from the purchase of the product (within product warranty period).
- ◆ In case the quality warranty sheet including the information requested above are not attached with the request for repair, free repair cannot be made.

[Paid service]

- ◆ For the failure occurred after 1 year of the purchase of the product (product warranty period), expenses can be charged for the repair.

- ◆ In case service is requested for the reason besides failure of the product, transportation cost, labor cost, etc. are charged. You are recommended to read product manual first and consult with dealer, the place you bought the product or manufacturer before asking service.

- ◆ In case failure occurs due to the fault of consumer as mentioned below, repair cost can be requested even within the product warranty period.

- . Failure occurs due to exposure to shock, moisture, high temperature, high voltage, etc. as a result of consumer mistake.
- . Failure occurs due to mishandling of the consumer.
- . In case there is evidence of arbitrarily repair, modification or disassembly.
- . In case failure happens by not complying the way of usage described in user manual.
- . In case the cause of the failure is not originated from the defect of the product.

- ◆ Paid service can be done even when quality warranty sheet is lost, and the responsibility for the loss of quality warranty sheet belongs to user.

[A / S] Contact point

